

## CLEANING THE PEDZ TAGS


The Pedz tag is not sterilized at the factory. You may wish to clean it before first use.

The Pedz tag can be cleaned using a soft-bristle brush. For minimal surface degradation and cracking over time, use the following guidelines:

- First, consult with your Infection Control representative for approved cleaners that can be used on plastics. Prolonged exposure with cleaners containing a high percentage (>60%) of isopropanol may cause some crazing of the tag's surface.

### Tools:

- ✓ Soft-bristle brush
- ✓ Hospital disinfectant cleaner designed for plastics
- ✓ Soft cloth or wipe

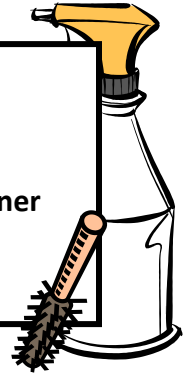

- Do not use an autoclave to clean the tag or serious damage may result.



1  
Spray a rag or use a disinfectant wipe to remove heavy soil on the surface.



2  
Spray the cleaner on the surface and any openings on the side of the tag or the removal key



3  
If needed, a soiled tag or removal key can be soaked in a cleaning solution



4  
If needed, use a brush to clean the tag surface and openings



5  
Rinse and dry thoroughly when finished