

HUGS TRAINING TEST A

Name: _____ Date: _____

- T or F** When a patient leaves the hospital, it is necessary to discharge the Hugs tag from the system (at the PC) before it is removed from the patient.
- T or F** The Hugs tag must be removed before giving a patient a bath. (tags are waterproof)
- T or F** The rounded part of the Hugs tag should face the patient's foot. (face knee)
- T or F** The Hugs tag should always be removed before the patient leaves the safe area. (should be Transported unless going to MRI or OR)
- T or F** A "heartbeat" is sent by the Hugs tag to ensure that it is working properly.
- T or F** The Hugs tag can be soaked for cleaning. (may activate and cause alarms)
- T or F** To save time, it is OK to put the strap in both ears of the Hugs tag (make a loop) and later slip it on the patient's ankle. (tags need skin contact or alarms will occur)
- T or F** The Hugs System should be explained to parents.
- T or F** If a Hugs tag is too tight, and a strap needs to be changed, you must discharge the tag and get both a new tag and a new strap. (use "Suspend Tamper" feature)

How tight should a Hugs tag be? It should wiggle but not rotate.

How often should you assess the Hugs tag? At least once every 8 hours (per shift)

When is the "Transport" function used? To allow the staff to take a patient with a tag outside of the "safe" area without causing an alarm.

When will a "Tamper Alarm" sound? When the strap on the tag is cut or pulled on.

What does the "BabySense™" feature do? Allows staff to be alerted when the tag is too loose so that they can tighten it before it gets loose enough to fall off.