

CLEANING THE HUGS TAGS

The Hugs tags are not sterilized at the factory. They will need to be cleaned before first use.

STANLEY
Healthcare


Technical Bulletin
Approved Cleaning Materials for Hugs® and Kisses® Tags

The Hugs tag can be cleaned using a soft-bristle brush. For minimal surface degradation and cracking over time, use the following guidelines:

- Use **ONLY** approved cleaning materials that the manufacturer has listed and follow manufacturer's cleaning instructions!
- Spray a cloth with approved disinfectant soap or similar solution and wipe off tag.
- May also use approved disinfectant wet wipe (be careful it is not too wet)
 - Do not spray the tags with water or cleaners.
 - Do not completely immerse the tags while cleaning.
 - If needed, use a soft-bristle brush.
 - If needed, gently clean the tag slots of strap debris.
- Wipe the tag with clear, clean water and dry the tag thoroughly before reuse.
 - Tags must rest for 10 minutes before using again.
 - Discharge and remove the tag before MRI scans and other medical procedures such as going to the operating room.
 - Tags may be left on during X-Ray if the tag is not in X-Ray field.

TOOLS:

- ✓ Soft-bristle brush
- ✓ Approved disinfectant cleaner
- ✓ Soft cloth

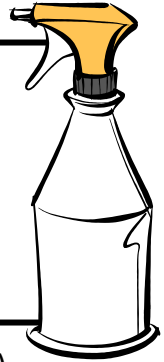

1
Spray a rag with an approved cleaner or use an approved disinfectant wipe to clean the tag



2
Wipe the tag with clear, clean water and dry thoroughly when finished cleaning



3
If needed, use a brush to clean the ears of the tag