

## HUGS TRAINING TAG



1. As the tag clearly defines – this is a “Training Tag.” It is never intended to be used on a real patient at any time. It will NOT communicate with a live Hugs System. It does not have a warranty for battery life.
2. Notice the word “STRAP” above this light and the word “ATTACHED” below the light. When the strap is put on the tag correctly, this light will turn on and remain red until the strap is cut or tampered with significantly. It is on the tag to show that the user has properly applied the strap. If it does not light up, either the strap is on incorrectly or something is wrong with the tag (most likely the battery has died).
3. Notice the words “ON BODY” above this light. This light comes on (red) when the tag’s BabySense feature is properly engaged. For this to occur, **both** the back of the tag and the strap must be touching “skin.” If the light is not on, the tag has either not engaged the skin tight enough (strap and back of tag) or something is wrong with the tag (most likely the battery has died and the “STRAP ATTACHED” light is also off).