


## PASSPORT SAMPLE ALARM NOTES BOX

**YOU CANNOT CLEAR AN ALARM UNTIL AN ALARM NOTE IS ENTERED**

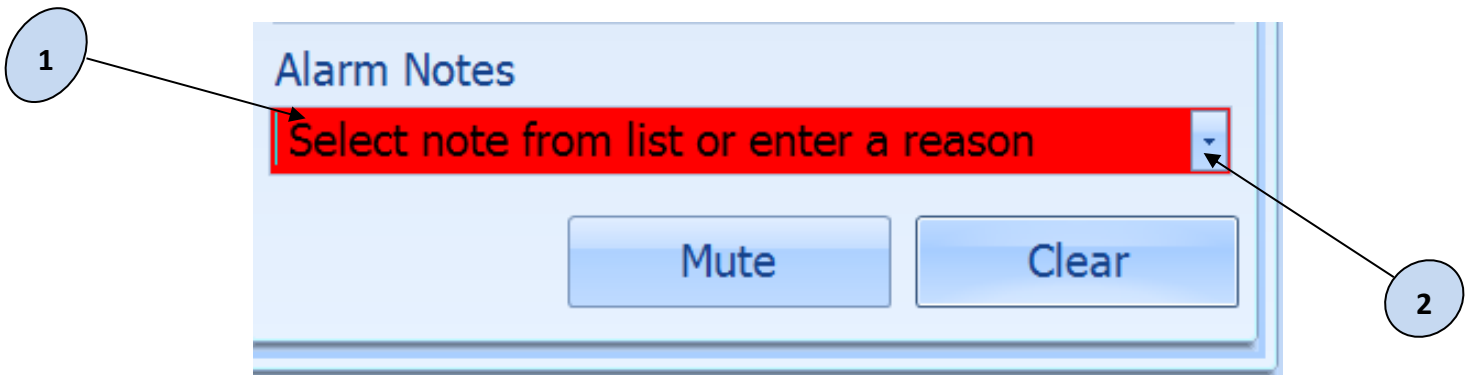

**1. You can type your own message here. Be brief, but be clear.**

**2. If you Click here, you will be given a list of pre-made messages to choose from:**

- Patient with tag got too close to the door.
- Patient with tag got too close to the Main Entrance Door.
- Forgot to Transport the tag before exiting the Department.
- Forgot to discharge the tag before exiting the Department.
- Tag came off patient. Will reapply.
- Removed the band before discharging the tag from the System.
- Discharged wrong tag. Will replace the band.
- Removed the band to change it before suspending the Tamper alarm.
- Tag in pocket with band pieces still attached.
- Family tried to loosen the tag. Will replace the band.
- Patient pulled the tag off. Will replace the band.
- Patient pulled on the tag. Will replace the band.
- Patient exercising. Will replace the band.
- Band not replaced yet. Notified nurse again.
- Patient safe in room. Will replace the tag with new one.
- Battery is low. Will give tag to System Administrator after discharge.
- Battery is low. Will replace as soon as possible.
- Door was being held open. Now closed.
- Door is not closing properly, called Maintenance Department.
- Too many incorrect Login attempts.
- Something was lying on the keyboard.
- Testing the system.
- Demonstration of the system.