


PEDZ DEFAULT ALARM NOTES

YOU CANNOT CLEAR AN ALARM UNTIL AN ALARM NOTE IS ENTERED

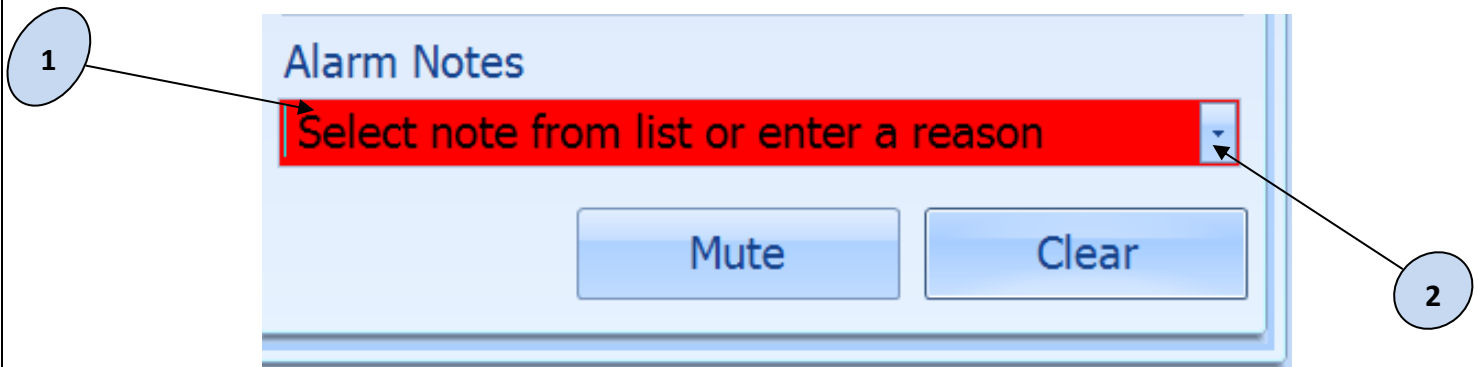

1. You can type your own message here. Be brief, but be clear.

2. If you Click here, you will be given a list of pre-made messages to choose from:

- Patient with tag got to close to the door.
- Patient with tag got to close to the Main Entrance Door.
- Forgot to Transport the tag before exiting the Department.
- Forgot to discharge the tag before exiting the Department.
- Tag is too loose. Will tighten.
- Tag came off patient. Will reapply.
- Cut the band before discharging the tag from the System.
- Discharged wrong tag. Will replace the band.
- Cut the band to change it before suspending the Tamper alarm.
- Tag in pocket with strap pieces still attached.
- Parent tried to loosen the tag. Will replace the Band.
- Parent pulled the tag off. Will replace the band.
- Parent cut the tag off before discharging tag from the System.
- Patient pulled on the tag. Will replace the band.
- Strap not replaced yet. Notified nurse again.
- Patient safe in room. Will replace the tag with new one.
- Cleaning the Tag, got too wet.
- Battery is low. Will give tag to Clinical Administrator after discharge.
- Battery is low. Will replace as soon as possible.
- Door was being held open. Now closed.
- Door is not closing properly, called Maintenance Department.
- Too many incorrect Login attempts.
- Something was lying on the keyboard.
- Testing the system.
- Demonstration of the system.