

Pedz Software “Alarms-Events” Matching Quiz

ALARMS-EVENTS

- | | |
|--|---------------------------------------|
| A. Exit Alarm | G. Low Battery Alarm |
| B. Tamper Alarm | H. Check Tag Tightness Event |
| C. Supervision Timeout Alarm | I. Loiter Event |
| D. Tag Loose Alarm | J. Door Ajar Event |
| E. Improperly Applied Tag Alarm | K. Admit Acknowledgement Event |
| F. Band Detached Alarm | |

Match the above “Alarms-Events” with their proper descriptions below. Place the letter next to the Alarm/Events above in the space next to its appropriate description below:

- _____ *Occur after a TAMPER ALARM or an IMPROPERLY APPLIED TAG have been cleared, but strap is still not securely attached to Tag, and the strap has not been changed and indicates that the strap is severely torn*
- _____ *Occur 10 minutes after an Admit Box has been on the screen without a description or zone*
- _____ *Occurs after a CHECK TAG TIGHTNESS EVENT and indicates that the tag is very loose and/or has possibly come off or has been taken off of the patient (only on Hugs Tag)*
- _____ *Occur when the system does not “hear” a Tag’s heartbeat. The Hugs PC can no longer see the tag.*
- _____ *Occur when Tag/patient comes near and/or passes through an open monitored door without being signed out of or discharged from the Pedz software*
- _____ *Occur when a monitored door has been open for pre-determined amount of time*
- _____ *Occur when the strap is not correctly attached to the Tag during the initial banding process*
- _____ *Occurs when the tag loses contact with the patient’s skin for more than 25 seconds (only on Hugs tag)*
- _____ *Occur when a Tag has been cut or pulled on in an effort to loosen or remove it*
- _____ *Occur when the Tags battery is becoming low*
- _____ *Occur when a Tag/patient has been near a monitored exit with a closed door for a pre-determined amount of time*