

HUGS TRAINING TEST B

Name: _____ Date: _____

ALARMS/EVENTS

- | | |
|------------------------------|---------------------------------|
| A. Exit Alarm | E. Improperly Applied Tag Alarm |
| B. Tamper Alarm | F. Check Tag Tightness Event |
| C. Supervision Timeout Alarm | G. Loiter Event |
| D. Tag Loose Alarm | H. Door Ajar Event |

Match the above Alarms/Events with their proper descriptions below. Place the letter next to the Alarm/Events above in the space next to its appropriate description below:

_____ *Occurs when the tag loses contact with the patient's skin for more than 25 seconds*

_____ *Occurs when the system does not "hear" a Tag's heartbeat; the Hugs PC can no longer see the tag*

_____ *Occurs when a patient with a tag comes near to and/or passes through an open monitored door or corridor without being transported or discharged from the Hugs software*

_____ *Occurs when a monitored door has been open for pre-determined amount of time*

_____ *Occurs after a CHECK TAG TIGHTNESS ALERT and indicates that the tag is very loose and/or has possibly come off or has been taken off of the patient*

_____ *Occurs when the strap is not correctly attached to the Tag during the initial banding process*

_____ *Occurs when a Hugs Tag has been cut or pulled on in an effort to loosen or remove it*

_____ *Occur when a Tag/patient has been near a monitored exit with a closed door for a pre-determined amount of time*