

Hugs Software Usage Quiz

Objective:

The purpose of these exercises is to affirm your familiarization with the Hugs system software and to check how you utilize it. Carefully read each question; and give a brief, clear answer in the space provided.

1. You put a Hugs tag on baby girl Smith in room 314. **A:** *Where do you enter the patient information on the computer?* **B:** *Clearly write out the patient information as you would type it into the computer(according to your hospital's policy):*
 - a) _____
 - b) _____
2. Baby Smith is moving from room 314 to room 360. **A:** *What needs to be done at the computer?*
 - a) _____

3. Baby Smith needs to leave the floor to go to ultrasound. She will be out of the department for 45 minutes. **A:** *What needs to be done at the computer?*
 - a) _____

4. Baby Smith has returned from ultrasound. **A:** *What needs to be done at the computer?*
 - a) _____

5. Baby Smith's parents think the Hugs strap is on too tight. You cannot pull the strap back because that will set off a tamper alarm; you cannot just cut the strap either. **A:** *What needs to be done at the computer before you cut the strap and put a new strap on?*
 - a) _____

6. Baby Smith is ready to go home. **A:** *What needs to be done at the computer before you remove the tag?*
 - a) _____

7. A **TAMPER** alarm is underway, and it has been determined that the infant is secure. The child's father thought the tag/strap was on too tight and pulled back on the tag! Before you can clear the alarm, the computer asks for a note about the alarm. **A:** *Clearly write out the appropriate note you might type into the computer or choose from the drop down menu.*
 - a) _____
