

## CORRECTIVE ACTIONS FOR DISMISSING ALERTS

### **EXIT**

Moved patient away from door.  
Properly transported patient.  
Properly discharged patient.  
Located patient & returned to Unit.  
Other

### **OUT OF UNIT**

Located patient & returned to Unit.  
Properly transported patient.  
Properly discharged patient.  
Other

### **TAG LOOSE**

Tightened the tag.  
Reapplied tag. Tag came off patient.  
Other

### **TAG TAMPER**

Replaced band which was too tight.  
Properly discharged tag.  
Reapplied tag & discharged correct patient.  
Replaced band damaged by caregiver.  
Notified nurse again, not replaced yet.  
Other

### **SUPERVISION**

Relocated patient within room.  
Replaced the tag with new one.  
Notified manager of repeated alarms.  
Other

### **BAND IMPROPERLY APPLIED**

Properly replaced band.  
Other

### **LOW BATTERY**

Replaced tag & put it in charger.

### **CHECK TAG TIGHTNESS**

Tightened tag immediately.

### **BAND DETACHED**

Applied a new band.  
Other

### **TAG LOITER**

Moved patient away from exit.  
Controller detecting tag in room.  
Other

### **INCOMPLETE TAG ADMIT**

Completed patient information.

### **TRANSPORT TIME OVERDUE WARNING**

Extended transport time.  
Return patient to Unit & cleared alert.  
Cleared alert. Patient in Unit.  
Other

### **TRANSPORT TIME OVERDUE RETURN**

Cleared alert. Patient in Unit.  
Return patient to Unit & cleared alert.  
Extended transport time.  
Other

### **DOOR AJAR**

Located open door, closed completely.

### **AUTO DISCHARGE**

Gave tag to manager, re-tagged patient.

### **DUPLICATE TAG**

Gave tag to manager, re-tagged patient.

### **TAG STILL BONDED**

Discharged tag. Used different tag.

### **MOTHER BABY MISMATCH**

Checked ID & corrected mismatch.  
Moved baby from incorrect mother.  
Other

### **KISSES LOW BATTERY**

Discharged Hugs tag. Re-tagged mom & baby.

**REASONS FOR SUSPENDING TAMPER**

Tag applied too tight. Changing band.  
Parent/Caregiver pulled tag to loosen it.  
Changing band.  
Replacing damaged band.  
Other

**REASONS FOR TRANSPORTING**

Transporting patient to radiology.  
Transporting patient for testing.  
Moving patient to new unit.  
Patient visiting mother out of unit.  
Other