

CLEANING THE PEDZ TAGS


The Pedz tag is not sterilized at the factory. You may wish to clean it before first use.

The Pedz tag can be cleaned using a soft-bristle brush. For minimal surface degradation and cracking over time, use the following guidelines:

- First, consult with your Infection Control representative for approved cleaners that can be used on plastics. Prolonged exposure with cleaners containing a high percentage (>60%) of isopropanol may cause some crazing of the tag's surface.

Tools:

- ✓ **Soft-bristle brush**
- ✓ **Hospital disinfectant cleaner designed for plastics**
- ✓ **Soft cloth or wipe**

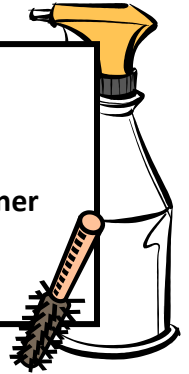

- Do not use an autoclave to clean the tag or serious damage may result.



1
Spray a rag or use a disinfectant wipe to remove heavy soil on the surface.



2
Spray the cleaner on the surface and any openings on the side of the tag or the removal key



3
If needed, a soiled tag or removal key can be soaked in a cleaning solution



4
If needed, use a brush to clean the tag surface and openings



5
Rinse and dry thoroughly when finished