


PEDZ SAMPLE ALARM NOTES BOX

YOU CANNOT CLEAR AN ALARM UNTIL AN ALARM NOTE IS ENTERED

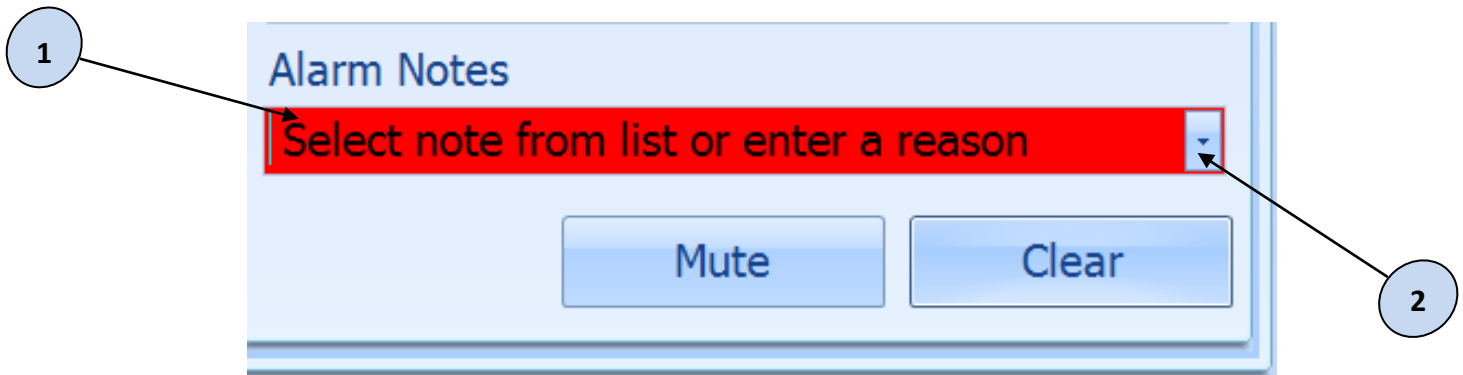

1. You can type your own message here. Be brief, but be clear.

2. If you Click here, you will be given a list of pre-made messages to choose from:

- Patient with tag got too close to the door.
- Patient with tag got too close to the Main Entrance Door.
- Forgot to Transport the tag before exiting the Department.
- Forgot to discharge the tag before exiting the Department.
- Tag is loose. Will tighten immediately.
- Tag came off patient. Will reapply.
- Removed the band before discharging the tag from the System.
- Discharged wrong tag. Will replace the band.
- Removed the band to change it before suspending the Tamper alarm.
- Tag in pocket with band pieces still attached.
- Family tried to loosen the tag. Will replace the band.
- Patient pulled the tag off. Will replace the band.
- Patient pulled on the tag. Will replace the band.
- Patient exercising. Will replace the band.
- Band not replaced yet. Notified nurse again.
- Patient safe in room. Will replace the tag with new one.
- Battery is low. Will give tag to System Administrator after discharge.
- Battery is low. Will replace as soon as possible.
- Door was being held open. Now closed.
- Door is not closing properly, called Maintenance Department.
- Too many incorrect Login attempts.
- Something was lying on the keyboard.
- Testing the system.
- Demonstration of the system.