

TEST FOR PEDZ SYSTEM OPERATION



Objective

The object of the test is to ensure that each participant is comfortable with the Pedz System and able to perform basic tasks using the software. It should be conducted in such a way as to build confidence and create a sense of achievement and ownership in the system. Any participant who fails in any of the test areas should be encouraged to try again until they succeed.

Method

These tests can be conducted on a PC with the live Pedz software loaded and running. Try to pick a time when activity with the computer is low. This method also works best in a 1 on 1 situation. Have the nurse apply a strap to a live tag to start the training. **IMPORTANT: Use the name "TEST" for the patient description/name.** Once admitted, use this tag to demonstrate each procedure. Once completed, discharge the tag. Evaluation should be based on the specific procedures for the facility. The Trainer may be required to role play during the tests. For example, when a Tag is admitted, the Trainer may want to take the role of the nurse attaching the Tag and relay the information about the infant to the user.

Materials Required

PC with the Pedz software loaded

Regular Procedures

Perform each of the following procedures. The trainer will evaluate your performance.

CRITERIA	1 st try	2 nd try
Log in to the system	Pass/Fail	Pass/Fail
Admit a Tag		
Proper procedure followed	Pass/Fail	Pass/Fail
Necessary information recorded	Pass/Fail	Pass/Fail
Transport a Tag		
Proper procedure followed	Pass/Fail	Pass/Fail
Necessary information recorded	Pass/Fail	Pass/Fail
Return a Tag from Transport		
Proper procedure followed	Pass/Fail	Pass/Fail
Necessary information recorded	Pass/Fail	Pass/Fail
Suspend Tamper Alarm for a Tag		
Proper procedure followed	Pass/Fail	Pass/Fail
Resume Tamper Alarm for a Tag		
Proper procedure followed	Pass/Fail	Pass/Fail
Change Patient's Name/Information		
Proper procedure followed	Pass/Fail	Pass/Fail
Necessary information recorded	Pass/Fail	Pass/Fail
Discharge a Tag		
Proper procedure followed	Pass/Fail	Pass/Fail
Necessary information recorded	Pass/Fail	Pass/Fail
Log out of the system	Pass/Fail	Pass/Fail